





Contour[®] next EZ
Blood Glucose Monitoring System

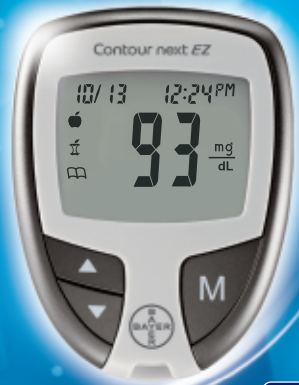

QUICK REFERENCE GUIDE

Your CONTOUR[®]NEXT EZ Meter

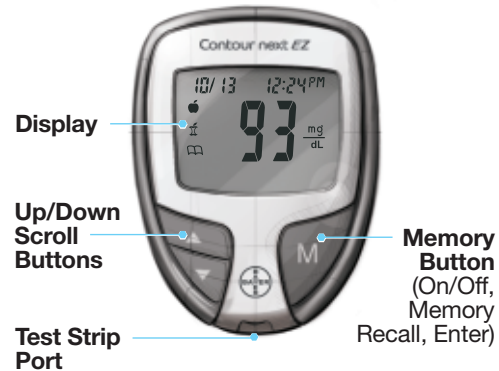

Use only the CONTOUR[®]NEXT Test Strip

Gray Square End: Insert this end into the test strip port.

Sample Tip: Blood sample pulled in here.



Basic Testing

Getting Ready to Test

1. Always wash your hands well with soap and water before and after handling the meter, lancing device, or test strips.

2. Insert the gray square end of the test strip into test strip port on the meter.



3. The meter will turn on and a test strip with a flashing blood drop will appear on your meter display letting you know the meter is ready to test.

4. Remove the endcap from the MICROLET[®]2 lancing device.



5. Loosen the round protective cap on a lancet by rotating it 1/4 turn, but do not remove it.



6. Insert the lancet firmly until it comes to a complete stop and re-sets the device.



7. Twist off the lancet cap.



8. Replace the endcap prior to getting your blood drop.




9. Adjust the endcap dial to your preferred puncture depth setting.



CAUTION: The CONTOUR[®]NEXT EZ meter and MICROLET[®]2 lancing device are intended for use by a single patient.

Getting the Blood Drop

1. Press the endcap firmly against the fingertip and press the blue release button with your thumb.

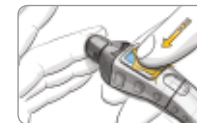

2. Stroke your hand and finger towards the puncture site to form a drop of blood.

Testing Your Blood

1. Use only CONTOUR[®]NEXT test strips with the CONTOUR[®]NEXT EZ meter.

2. Touch the tip of the test strip to the drop of blood. The blood is pulled into the test strip.

3. Hold the strip in the blood drop until the meter beeps.

4. Your result is displayed after a 5 second countdown.

5. To turn your meter off, remove the test strip. Dispose of the test strip as medical waste.



Fingertip Testing



See your CONTOUR[®]NEXT EZ User Guide and MICROLET[®]2 insert for complete instructions.

Removing the used lancet

1. Hold the lancing device with your thumb on the grip indent and the endcap as shown. Gently snap off the endcap.



2. Place the round protective lancet cap on a flat surface with the Bayer logo facing down.



3. With the used lancet still in the lancing device, push the lancet needle completely into the middle of the exposed side of the cap.

4. Eject the lancet into a container following the instructions in your lancing device insert.

5. Replace the endcap.

Need Help? 1-800-348-8100
(available 24 hours a day, 7 days a week)
www.bayercontour.com

Potential Biohazard

All blood glucose measuring systems are considered biohazardous.

All products or objects which come in contact with human blood, even after cleaning, should be handled as if capable of transmitting viral diseases.

The meter and lancing device are for single patient use. Do not share them with anyone including other family members!

All parts of the kit are considered biohazardous and can potentially transmit infectious diseases, even after you have performed cleaning and disinfection.

Always wash your hands well with soap and water before and after handling the meter, lancing device, or test strip.

See your user guide for complete instructions.

Using Advanced Mode

Setting Your Meter to Advanced Mode

To set your meter to Advanced Mode, and take advantage of meal markers and other available features:




1. Press **M** to turn your meter on.
2. Press and hold **▲** or **▼** for 3 seconds.
3. L-1 will flash.
4. Press **▲** or **▼** to change to L-2.
5. Press **M** to set.




Find out how these additional features can help you manage your diabetes.

Contact Bayer Diabetes Care
Customer Service at 1-800-348-8100
(available 24 hours a day, 7 days a week)
www.bayercontour.com

After you perform a test and see your results, use the markers on the display:


To Mark Results >	You Will See >	To Set >	You Will See >	What It Means >
Pre-Meal Marks a test result that was taken before a meal. Press ▲ or ▼	 (Flashing)	Press M		Result is marked as a pre-meal test. Press M again to set a reminder to run a post-meal test.
Post-Meal Marks a test result that was taken after a meal. Press ▲ or ▼	 (Flashing)	Press M		Result is marked as a post-meal test.
Logbook Marks any result as being unique or different in some way. Press ▲ or ▼	 (Flashing)	Press M		Result is marked in memory as unique. You should note this in your logbook.

 Sets a reminder (pre-set at 2 hours) to run a post-meal test. See user guide for complete instructions on how to use the markers and adjust the reminder times

Using The Reminder




When the reminder sounds, the meter turns on, and you will see your pre-meal test result.



Press any button once to silence reminder and twice to turn the meter off. Run another blood test and mark it with a  post-meal marker (see chart at left).

Error Codes and Symbols

See your user guide for a complete list of Error Codes.

E1 	Temperature out of range.
E2 	There was a problem completely filling the test strip. Retest using a new test strip.
E3 	The meter is sensing a used test strip or the wrong control solution was used. Retest using a new test strip.
E11	Abnormal result. Retest using a new test strip.

 **Bayer HealthCare**
Bayer HealthCare LLC
Mishawaka, IN 46544 USA
www.bayercontour.com

Bayer
83791935
Rev. 02/12